




NOVAKOR  
GLOBAL

2.5 mL x 1

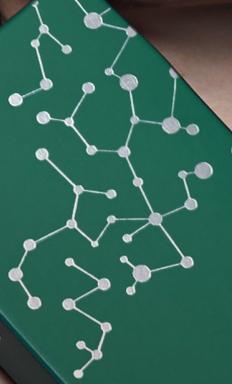

HILO 32

High molecule HA  
Low molecule HA

HILO 32

HIGH AND LOW MOLECULAR  
WEIGHT HA: THE PERFECT DUO  
FOR SKIN PERFECTION

[www.novakorglobal.com](http://www.novakorglobal.com)

Made in Korea

# HIGH MOLECULAR HA + LOW MOLECULAR HA



HILO 32



## Why Hilo 32?

✓ Lifting

✓ Elasticity Improvement

✓ Returns Volume

✓ Balance Recovery

✓ Restoring Hydration

✓ Skin Regeneration

✓ Stimulate Collagen Synthesis


✓ Anti-Inflammatory

✓ Tissue Remodeling

✓ Skin Tightening


# MOLECULAR WEIGHT

H-HA

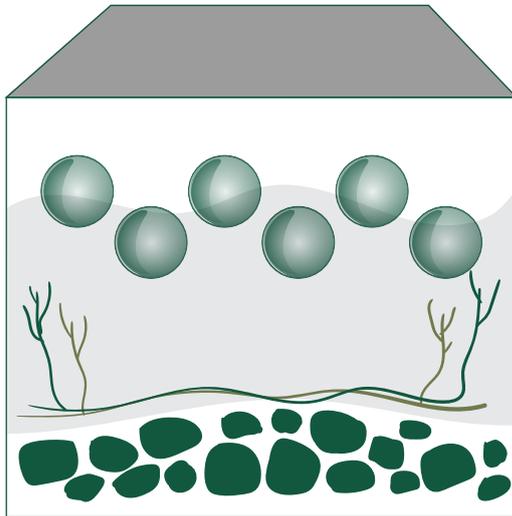

The High molecular weight hyaluronic acid increases skin volume, delivering a noticeable lifting effect.

L-HA

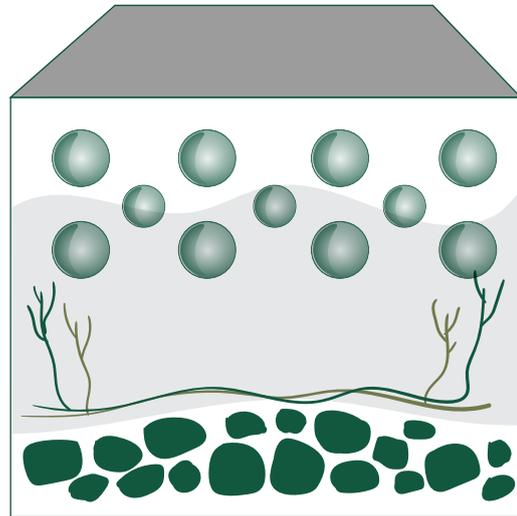

The Low molecular weight hyaluronic acid penetrates deeply, delivering intensive hydration to the epidermis.

**THIS NON CROSS-LINKED HYALURONIC ACID SKIN BOOSTER IS DEVELOPED WITH CUTTING-EDGE TECHNOLOGY TO IMPROVE WRINKLES.**

Utilizing a proprietary high-concentration manufacturing method, it overcomes the limitations of traditional low-elasticity (monophasic) and low-viscosity (biphasic) HA formulations. This makes it highly effective for both wrinkle treatment and facial contour enhancement.



# HILO 32

Non cross-linked H-HA & L-HA


Package: 2.5 ml per syringe

Does not include needle

Hilo 32 excels in providing top-tier hydration and lifting effects, making it an excellent choice for individuals seeking immediate plumping and firming of the skin. Its high effectiveness in improving skin elasticity also contributes to a more youthful appearance.

However, it offers moderate benefits in anti-inflammatory properties and skin regeneration, making it suitable for those primarily focused on hydration and lifting.

**Injection Depth:** Epidermis, Dermis

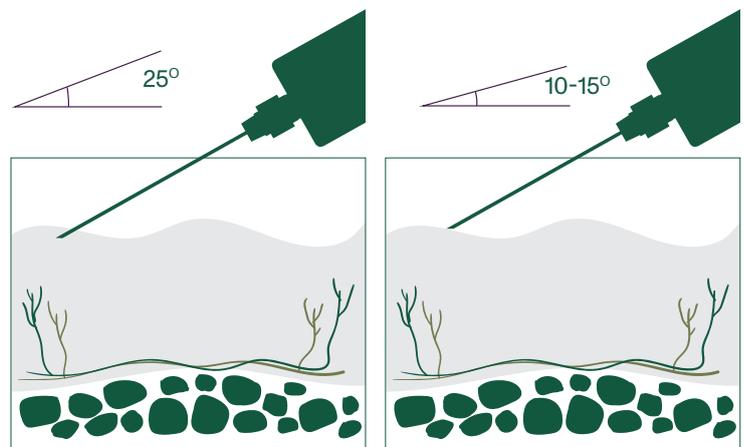

RECOMMENDED NUMBER OF TREATMENTS

2 - 3

INTERVAL BETWEEN TREATMENTS:

1 MONTH

SHELF LIFE

36 MONTHS

STORE AT ROOM TEMPERATURE, AVOID DIRECT SUNLIGHT

(1-30°C)

HYDRATION



LIFTING



VOLUME



ELASTICITY



BALANCE RECOVERY



SKIN REGENERATION



↑ 40 mg of High Molecular HA

↑ 40 mg of Low Molecular HA





2.5 mL x 1

HILO 32

2.5 mL x 1

HILO 32

2.5 mL x 1

HILO 32

High molecule HA  
Low molecule HA

# FACIAL REMODELING


## 6 POINTS BAP TECHNIQUE

With just 6 BAP injection points per side of the face, Hilo32 diffuses through multiple skin layers, covering broad areas of the face, and delivering a natural and long-lasting refreshed look.




Recommended Needle size	27G / 30G
Product quantity per Point	0.2 ml
Product quantity per Treatment	2.4 ml
Injection depth	1.5 - 4 mm
Angle of Inclination	0 - 25°
Interval between treatment	4 weeks.
Frequency	2 -3 times per year
Avoid blood vessel locations	

# Facial BAP Technique Measurements

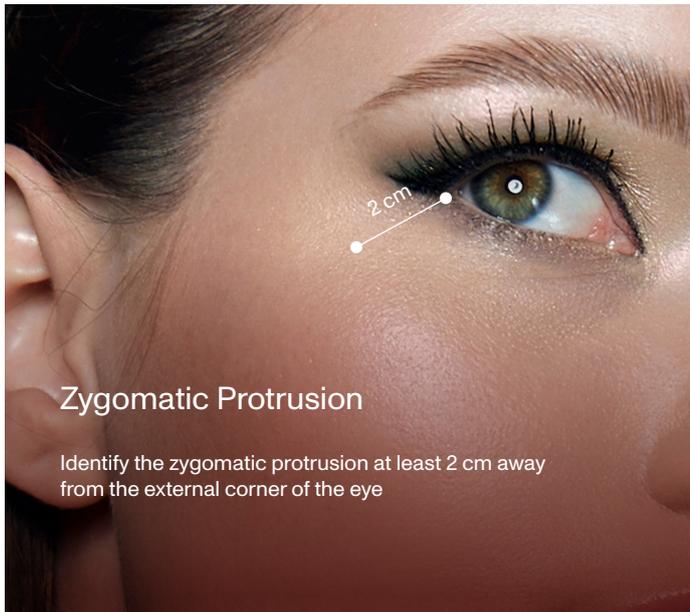


## Zygomatic Protrusion

Identify the zygomatic protrusion at least 2 cm away from the external corner of the eye

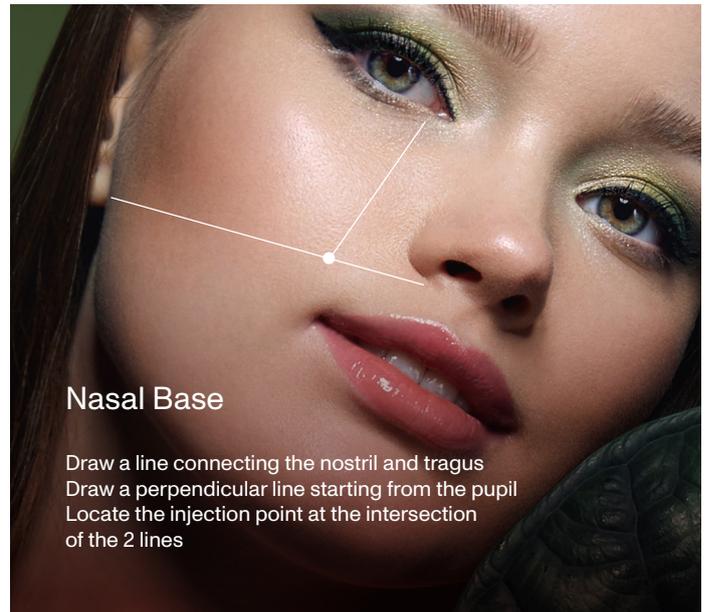


## Nasal Base

Draw a line connecting the nostril and tragus  
Draw a perpendicular line starting from the pupil  
Locate the injection point at the intersection of the 2 lines

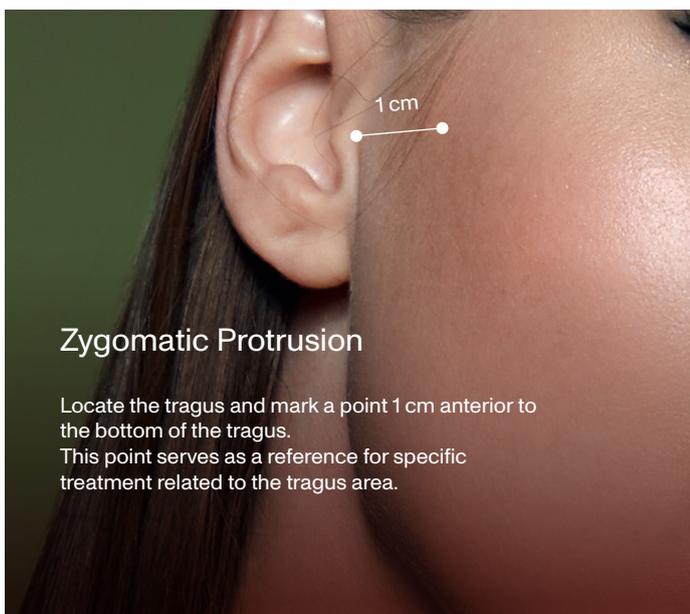


## Zygomatic Protrusion

Locate the tragus and mark a point 1 cm anterior to the bottom of the tragus.  
This point serves as a reference for specific treatment related to the tragus area.

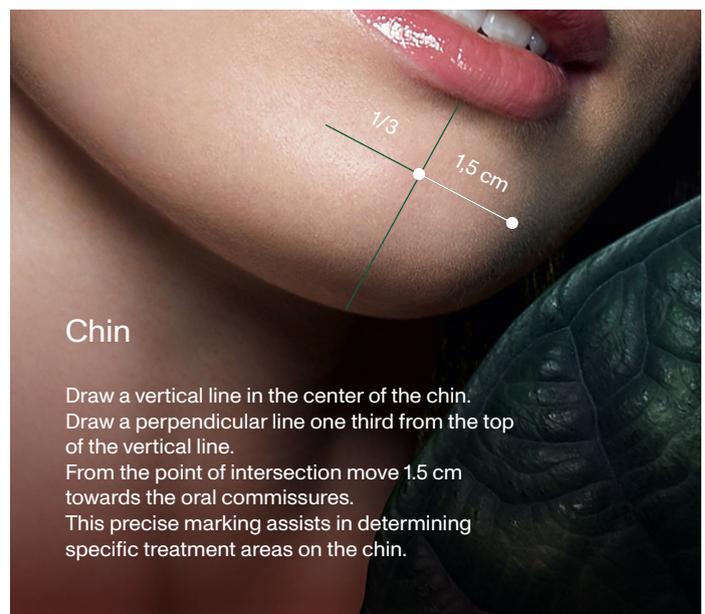


## Chin

Draw a vertical line in the center of the chin.  
Draw a perpendicular line one third from the top of the vertical line.  
From the point of intersection move 1.5 cm towards the oral commissures.  
This precise marking assists in determining specific treatment areas on the chin.

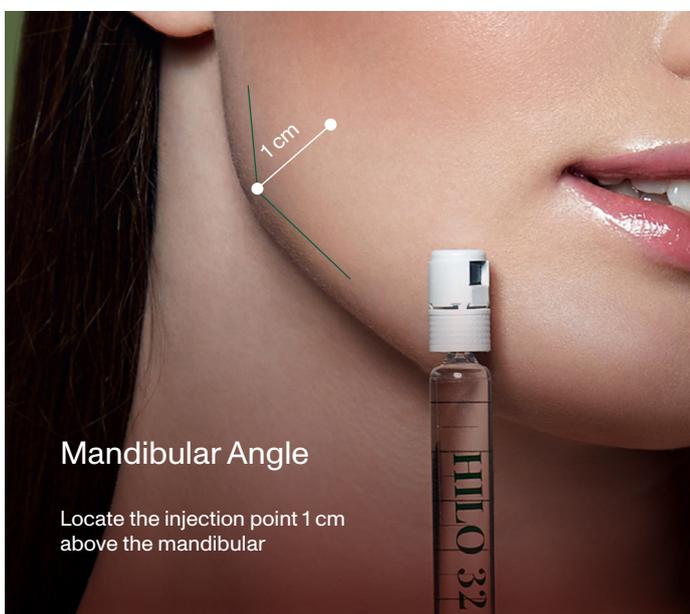

## Mandibular Angle

Locate the injection point 1 cm above the mandibular

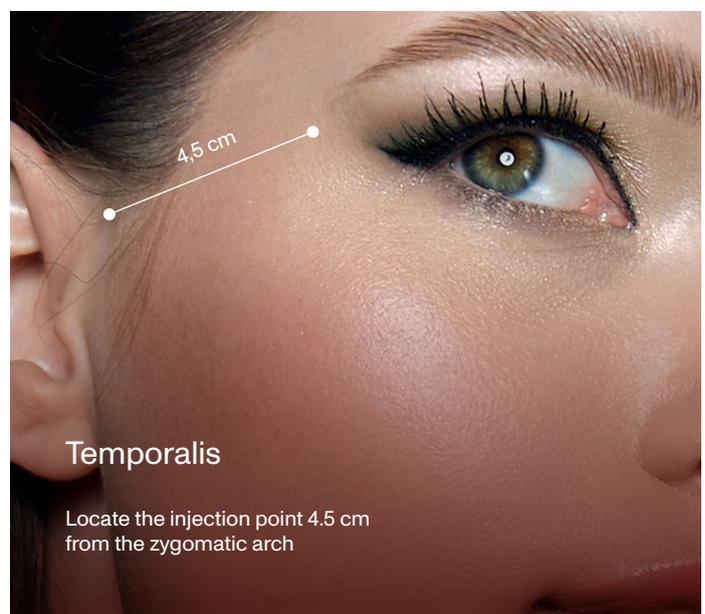


## Temporalis

Locate the injection point 4.5 cm from the zygomatic arch

# NECK REMODELING 12 POINTS BAP TECHNIQUE

The 12 BAP injection points for the neck are carefully designed to avoid injury to vital structures, delivering anti-aging benefits from the inside out.

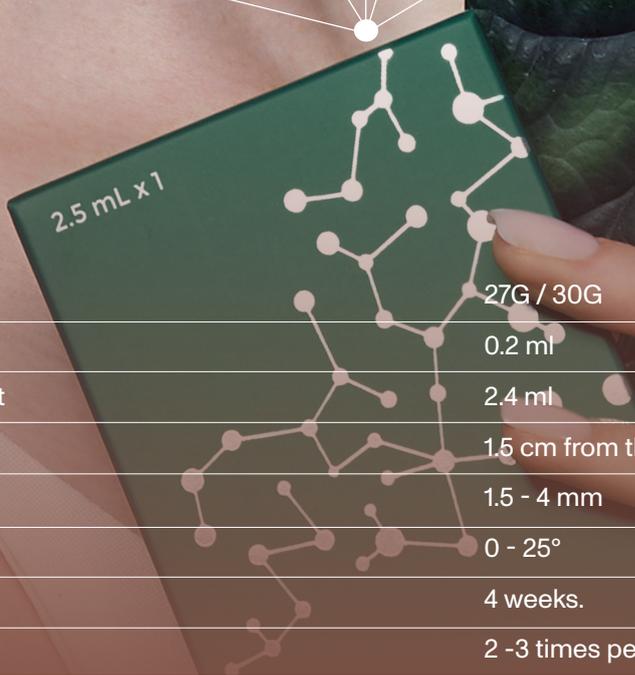

Recommended Needle size	27G / 30G
Product quantity per Point	0.2 ml
Product quantity per Treatment	2.4 ml
Distance	1.5 cm from the center of the clavicle
Injection depth	1.5 - 4 mm
Angle of Inclination	0 - 25°
Interval between treatment	4 weeks.
Frequency	2 -3 times per year
Avoid blood vessel locations	

# HAND REMODELING FANNING 10 POINTS BAP TECHNIQUE


The 10 BAP injection points for the hands are meticulously placed to ensure safety and effectiveness, delivering comprehensive anti-aging benefits and rejuvenation from within.



Recommended Needle size	27G / 30G
Product quantity per Point	0.2 ml
Product quantity per Treatment	4 ml ( both hands )
Distance	1.5 cm from the middle of the middle finger
Injection depth	1.5 - 4 mm
Angle of Inclination	0 - 25°
Interval between treatment	4 weeks.
Frequency	2 -3 times per year
Avoid blood vessel locations	

# INNER ARM 7 POINTS BAP TECHNIQUE

With 7 carefully selected BAP injection points, the inner arm receives a concentrated treatment that promotes skin rejuvenation and smoothness, effectively addressing signs of aging in this delicate area.

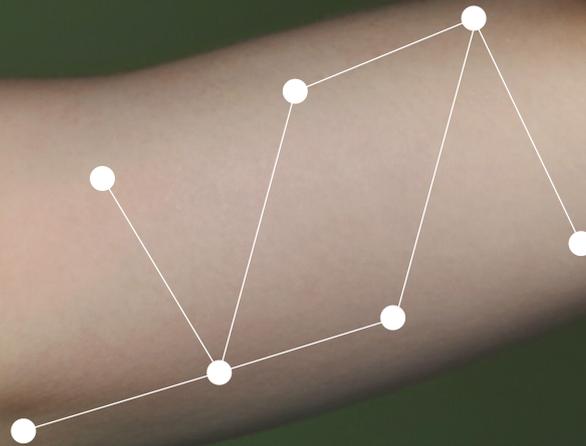

Recommended Needle size	27G / 30G
Product quantity per Point	0.2 ml
Product quantity per Treatment	3 ml ( both arms )
Injection depth	1.5 - 4 mm
Angle of Inclination	0 - 25°
Interval between treatment	4 weeks.
Frequency	2 -3 times per year
Avoid blood vessel locations	




HILO 32

HIGH AND LOW  
MOLECULAR WEIGHT HA

The Perfect Duo for Skin Perfection

Made in Korea

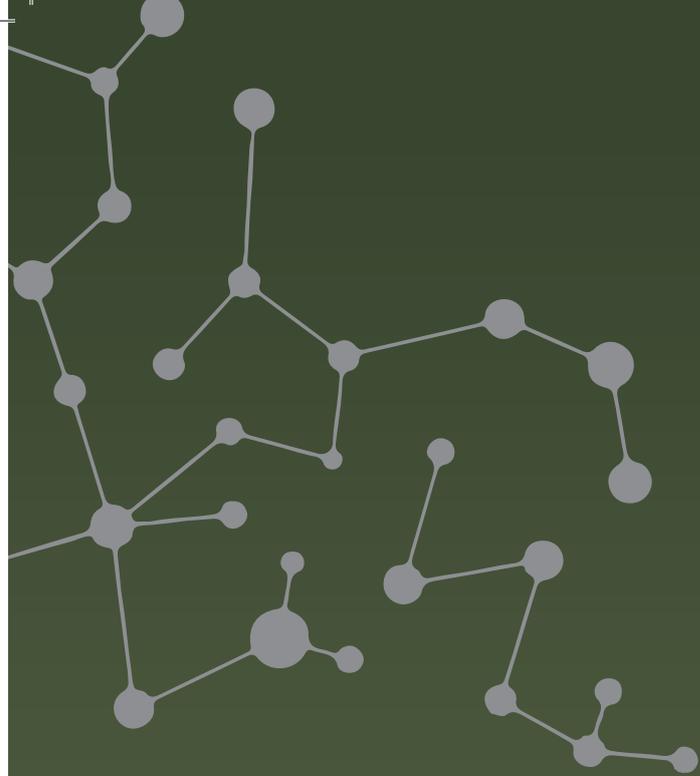

NOVAKOR  
GLOBAL



[novakorglobal.com](http://novakorglobal.com)



[info@novakorglobal.com](mailto:info@novakorglobal.com)

Made in Korea