



NOVAKOR
GLOBAL



HILO 32
Plus

HIGH AND LOW MOLECULAR
WEIGHT HA: THE PERFECT DUO
FOR SKIN PERFECTION

www.novakorglobal.com

Made in Korea

HIGH MOLECULAR HA LOW MOLECULAR HA SUCCINIC ACID PEPTIDES

HILO 32
Plus



Why Hilo 32 Plus?

✓ Lifting

✓ Elasticity Improvement

✓ Returns Volume

✓ Balance Recovery

✓ Restoring Hydration

✓ Skin Regeneration

✓ Stimulate Collagen Synthesis


✓ Anti-Inflammatory

✓ Tissue Remodeling

✓ Skin Tightening


MOLECULAR WEIGHT

H-HA

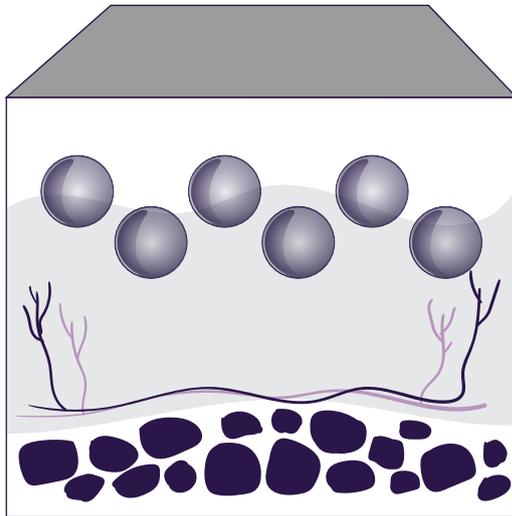

The High molecular weight hyaluronic acid increases skin volume, delivering a noticeable lifting effect.

L-HA

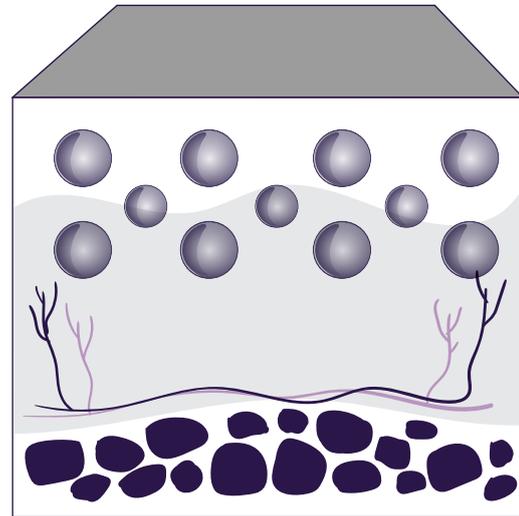

The Low molecular weight hyaluronic acid penetrates deeply, delivering intensive hydration to the epidermis.

THIS NON CROSS-LINKED HYALURONIC ACID SKIN BOOSTER IS DEVELOPED WITH CUTTING-EDGE TECHNOLOGY TO IMPROVE WRINKLES.

Utilizing a proprietary high-concentration manufacturing method, it overcomes the limitations of traditional low-elasticity (monophasic) and low-viscosity (biphasic) HA formulations. This makes it highly effective for both wrinkle treatment and facial contour enhancement.

HILO 32 Plus

Non cross-linked H-HA & L-HA

Package: 2.5 ml per syringe

Does not include needle

Hilo 32 Plus is designed for comprehensive mesotherapy, providing excellent benefits across multiple areas.

While its hydration and lifting effects are strong, its true strengths lie in enhancing skin elasticity, reducing inflammation, and promoting skin regeneration.

This makes it ideal for users looking for a well-rounded product that addresses various skin concerns, including aging and sensitivity.



RECOMMENDED NUMBER OF TREATMENTS

2 - 3

INTERVAL BETWEEN TREATMENTS:

1 MONTH


SHELF LIFE

36 MONTHS

STORE AT ROOM TEMPERATURE, AVOID DIRECT SUNLIGHT

(1-30°C)

Injection Depth: Epidermis, Dermis

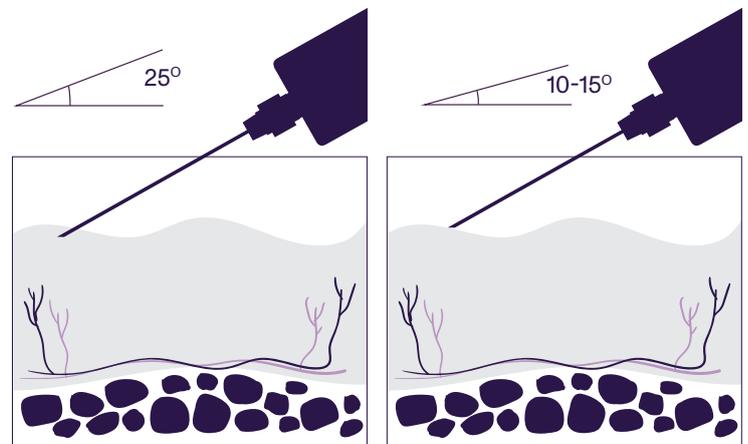

HYDRATION



LIFTING



VOLUME



ELASTICITY



BALANCE RECOVERY



SKIN REGENERATION



↑ 40 mg of High Molecular HA

↑ 40 mg of Low Molecular HA




↑ 40 mg Succinic Acid Peptide








FACIAL REMODELING 6 POINTS BAP TECHNIQUE

With just 6 BAP injection points per side of the face, Hilo32 diffuses through multiple skin layers, covering broad areas of the face, and delivering a natural and long-lasting refreshed look.

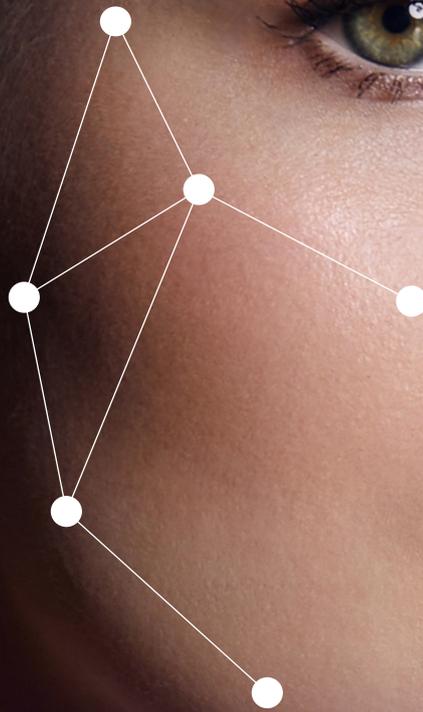


Recommended Needle size	27G / 30G
Product quantity per Point	0.2 ml
Product quantity per Treatment	2.4 ml
Injection depth	1.5 - 4 mm
Angle of Inclination	0 - 25°
Interval between treatment	4 weeks.
Frequency	2 -3 times per year
Avoid blood vessel locations	

Facial BAP Technique Measurements

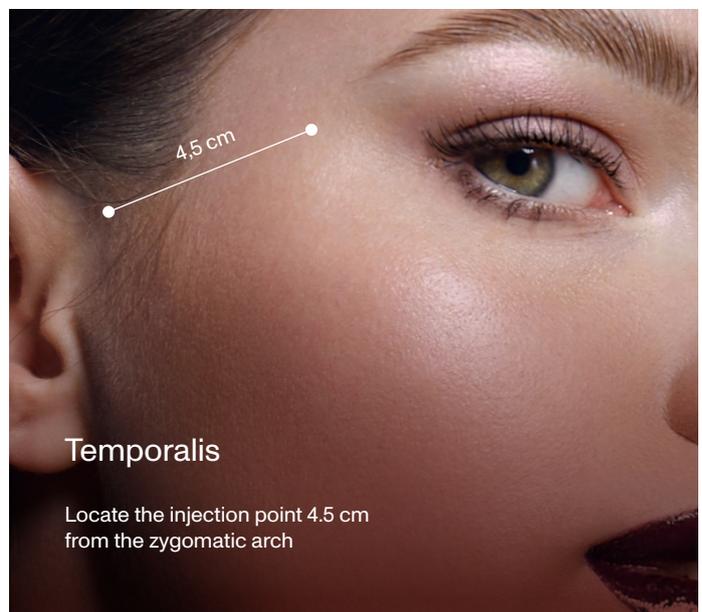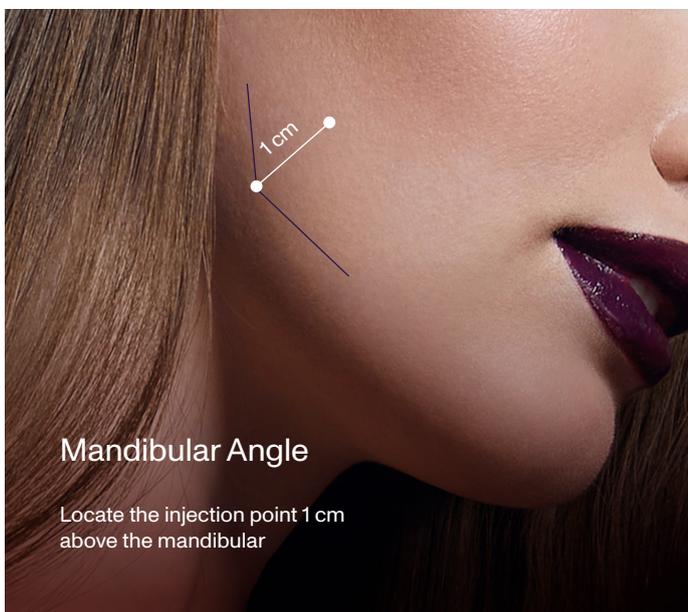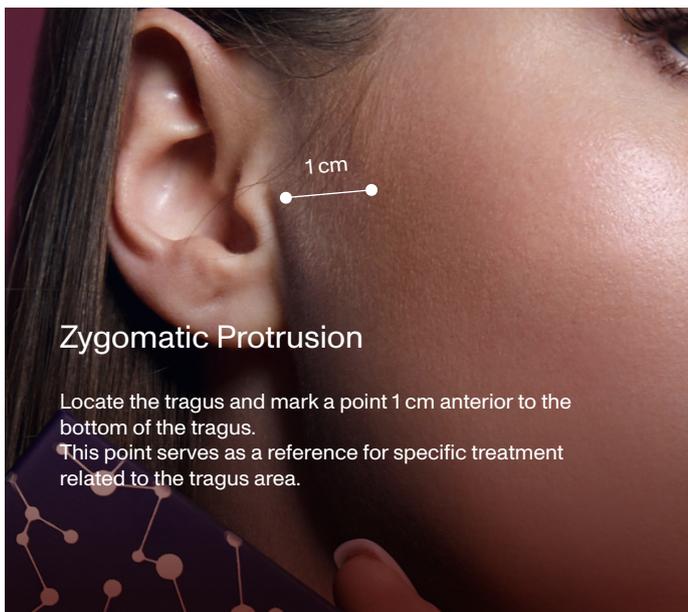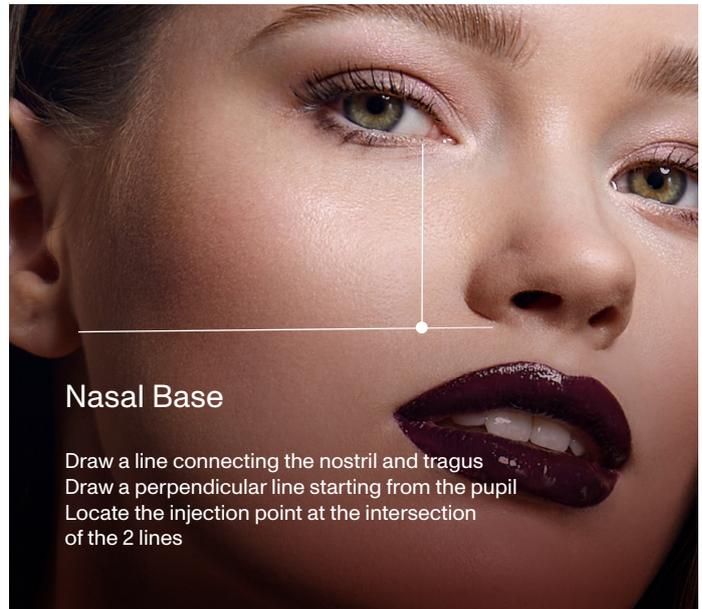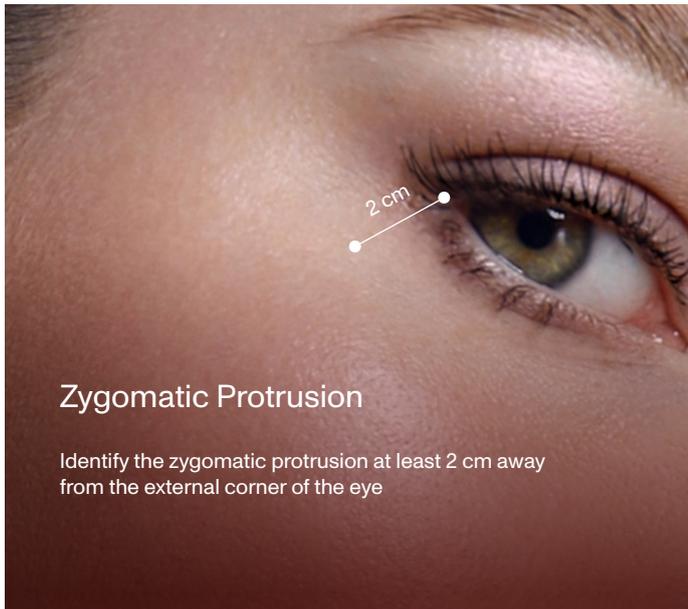

NECK REMODELING 12 POINTS BAP TECHNIQUE

The 12 BAP injection points for the neck are carefully designed to avoid injury to vital structures, delivering anti-aging benefits from the inside out.

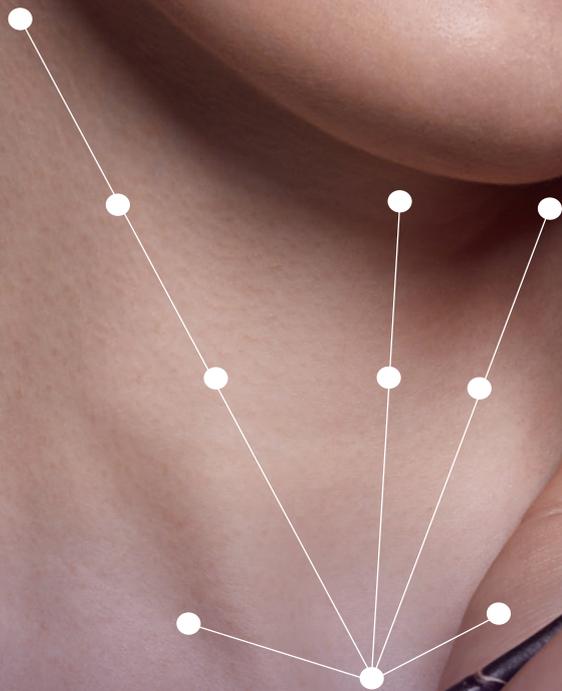

Recommended Needle size	27G / 30G
Product quantity per Point	0.2 ml
Product quantity per Treatment	2.4 ml
Distance	1.5 cm from the center of the clavicle
Injection depth	1.5 - 4 mm
Angle of Inclination	0 - 25°
Interval between treatment	4 weeks.
Frequency	2 -3 times per year
Avoid blood vessel locations	

HAND REMODELING FANNING 10 POINTS BAP TECHNIQUE

The 10 BAP injection points for the hands are meticulously placed to ensure safety and effectiveness, delivering comprehensive anti-aging benefits and rejuvenation from within.




Recommended Needle size	27G / 30G
Product quantity per Point	0.2 ml
Product quantity per Treatment	4 ml (both hands)
Distance	1.5 cm from the middle of the middle finger
Injection depth	1.5 - 4 mm
Angle of Inclination	0 - 25°
Interval between treatment	4 weeks.
Frequency	2 -3 times per year
Avoid blood vessel locations	

INNER ARM

7 POINTS BAP TECHNIQUE

With 7 carefully selected BAP injection points, the inner arm receives a concentrated treatment that promotes skin rejuvenation and smoothness, effectively addressing signs of aging in this delicate area.

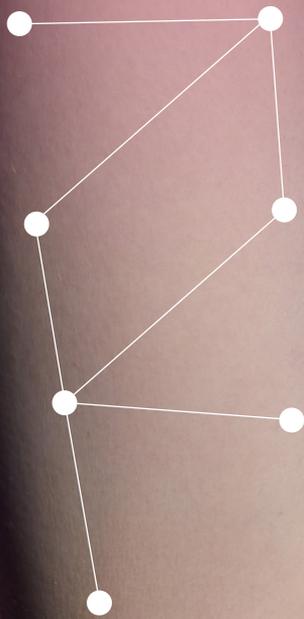

Recommended Needle size	27G / 30G
Product quantity per Point	0.2 ml
Product quantity per Treatment	3 ml (both arms)
Injection depth	1.5 - 4 mm
Angle of Inclination	0 - 25°
Interval between treatment	4 weeks.
Frequency	2 -3 times per year
Avoid blood vessel locations	

High molecule HA
Low molecule HA
succinic acid
peptide

HILO 20

2.5 mL x 1

2.5 mL x 1




HILO 32 Plus

HIGH AND LOW
MOLECULAR WEIGHT HA
SUCCINIC ACID
PEPTIDES

The Perfect Duo for Skin Perfection

Made in Korea

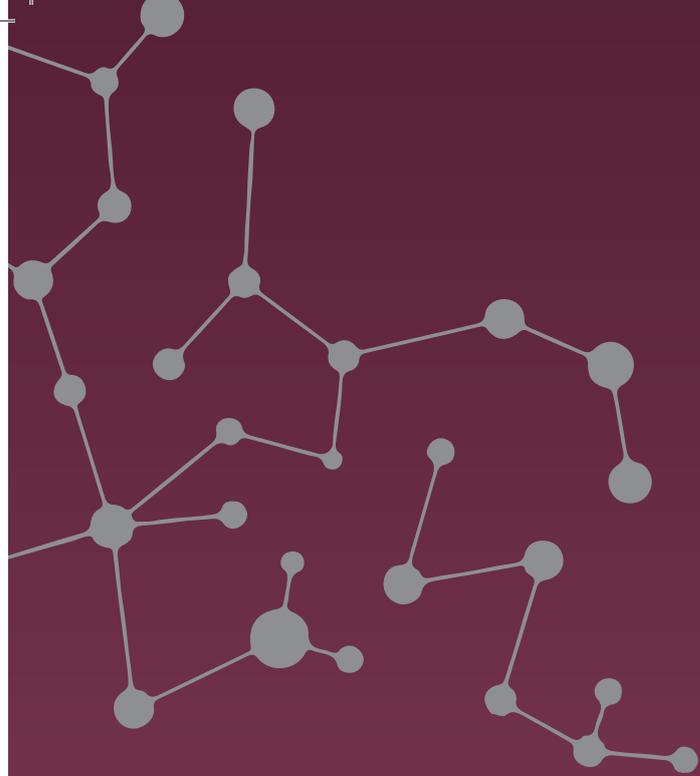

NOVAKOR
GLOBAL



novakorglobal.com



info@novakorglobal.com

Made in Korea