






NOVAKOR
GLOBAL

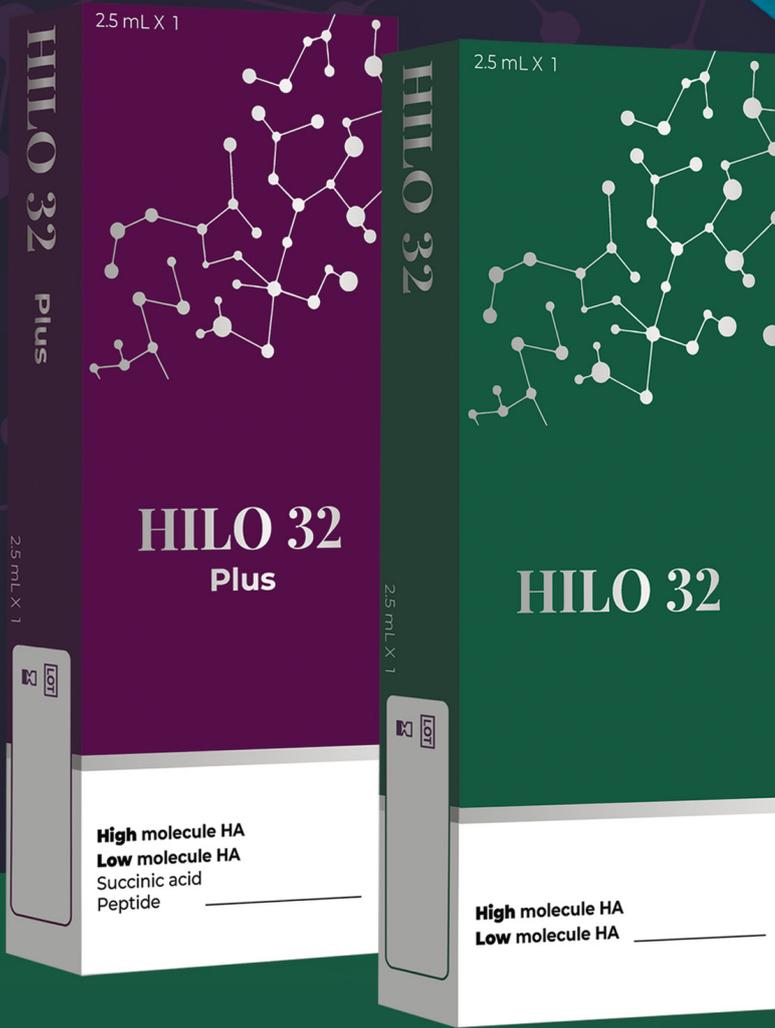

HIGH AND LOW MOLECULAR WEIGHT HA: THE PERFECT DUO FOR SKIN PERFECTION

**HIGH
MOLECULAR
HA + LOW
MOLECULAR
HA**

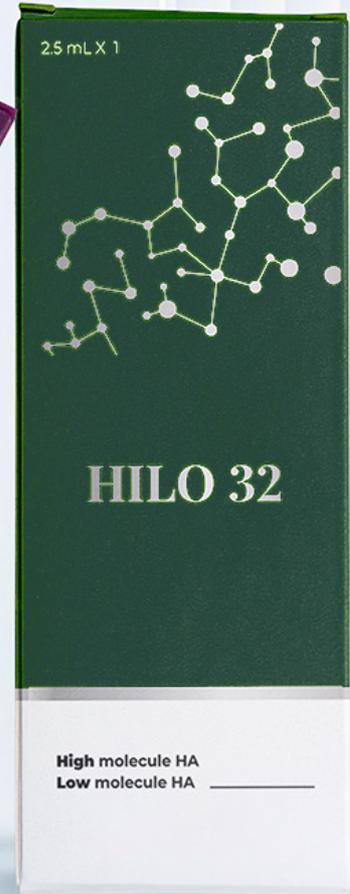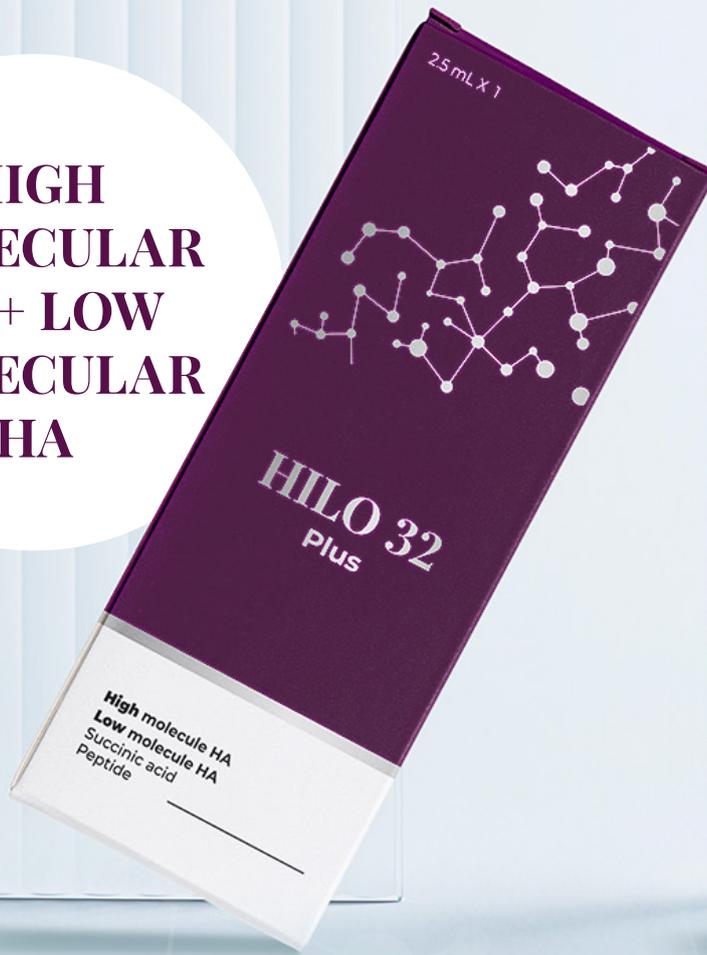

WHY HILO 32 /HILO 32 PLUS?



Lifting



Elasticity Improvement



Returns Volume



Balance Recovery



Restoring Hydration



Skin Regeneration



Stimulate Collagen Synthesis




Anti-Inflammatory



Tissue Remodeling



Skin Tightening

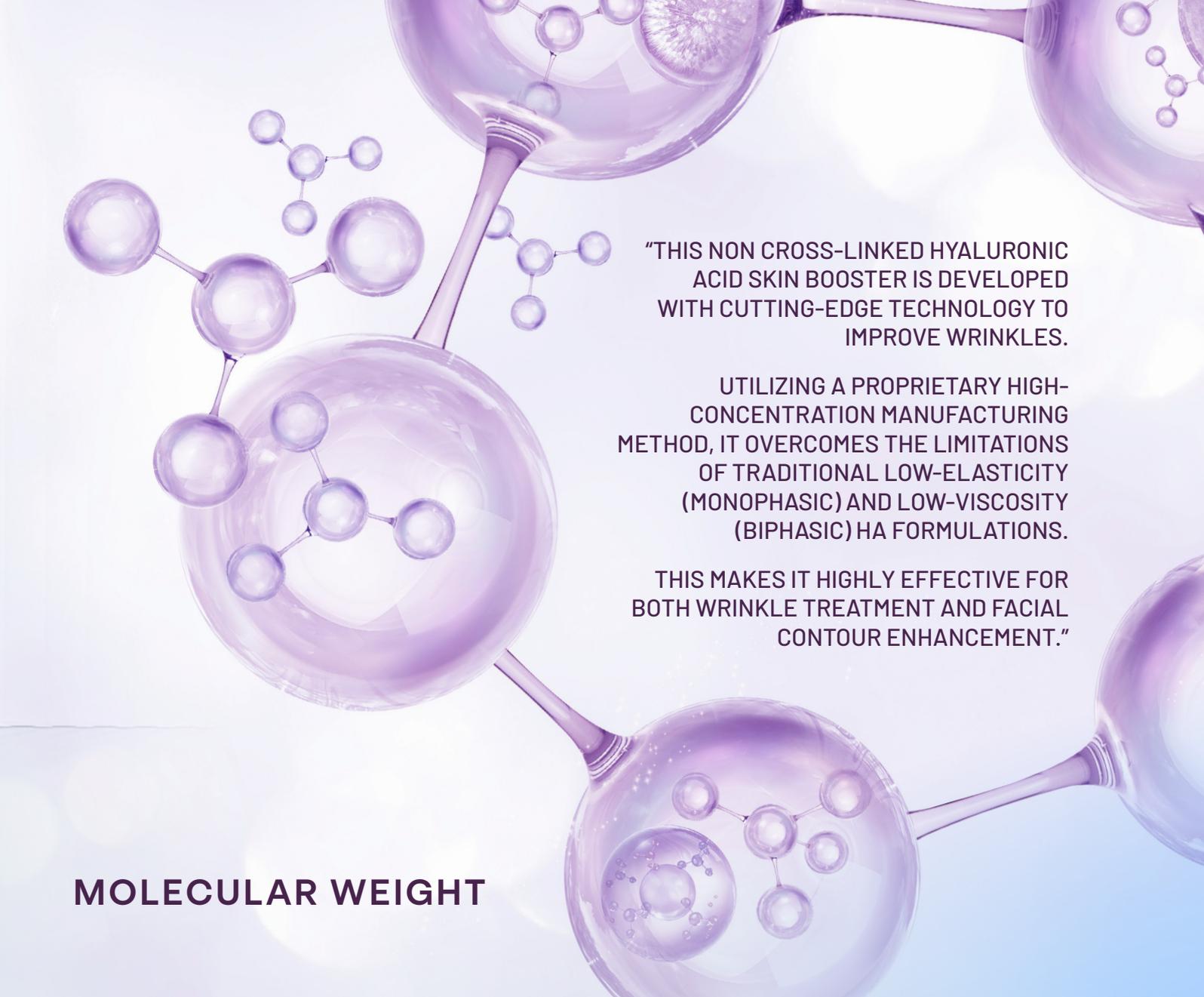


"THIS NON CROSS-LINKED HYALURONIC ACID SKIN BOOSTER IS DEVELOPED WITH CUTTING-EDGE TECHNOLOGY TO IMPROVE WRINKLES.


UTILIZING A PROPRIETARY HIGH-CONCENTRATION MANUFACTURING METHOD, IT OVERCOMES THE LIMITATIONS OF TRADITIONAL LOW-ELASTICITY (MONOPHASIC) AND LOW-VISCOSITY (BIPHASIC) HA FORMULATIONS.

THIS MAKES IT HIGHLY EFFECTIVE FOR BOTH WRINKLE TREATMENT AND FACIAL CONTOUR ENHANCEMENT."


MOLECULAR WEIGHT

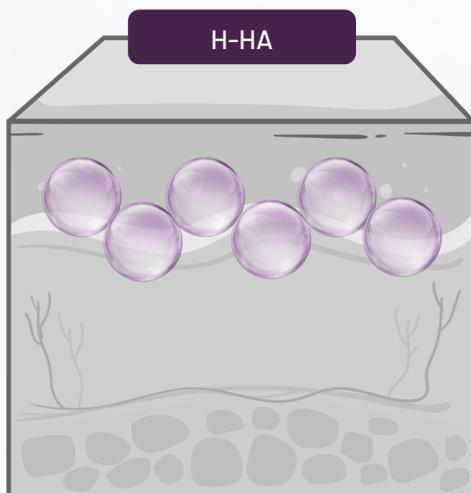

THE HIGH MOLECULAR WEIGHT HYALURONIC ACID INCREASES SKIN VOLUME, DELIVERING A NOTICEABLE LIFTING EFFECT.

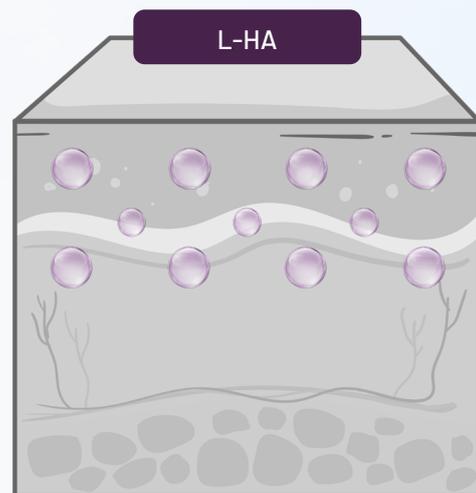

THE LOW MOLECULAR WEIGHT HYALURONIC ACID PENETRATES DEEPLY, DELIVERING INTENSIVE HYDRATION TO THE EPIDERMIS.

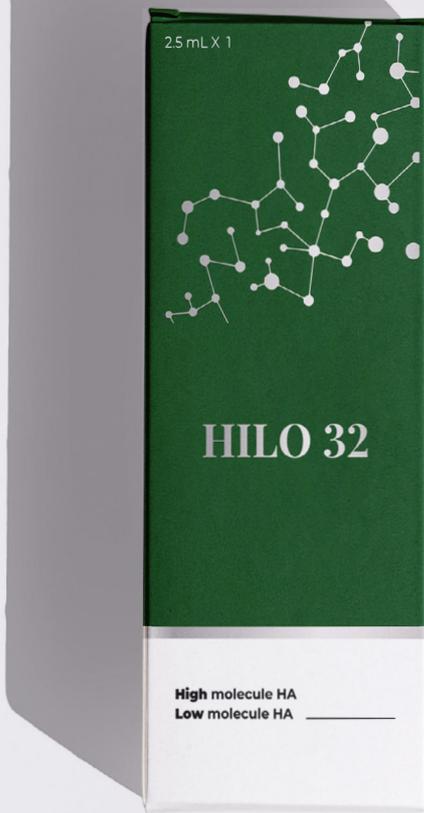

HILO 32

NON CROSS-LINKED H-HA & L-HA

Package: 2.5 ml per syringe

Does not include needle

Hilo 32 excels in providing top-tier hydration and lifting effects, making it an excellent choice for individuals seeking immediate plumping and firming of the skin.

Its high effectiveness in improving skin elasticity also contributes to a more youthful appearance.

However, it offers moderate benefits in anti-inflammatory properties and skin regeneration, making it suitable for those primarily focused on hydration and lifting.

Hydration



Lifting



Volume



Elasticity



Balance Recovery



Skin Regeneration



Composition:



40 mg of
High Molecular HA

40 mg of
Low Molecular HA



Injection Depth:

Epidermis, Dermis

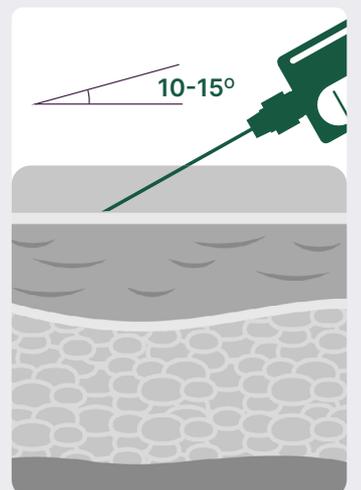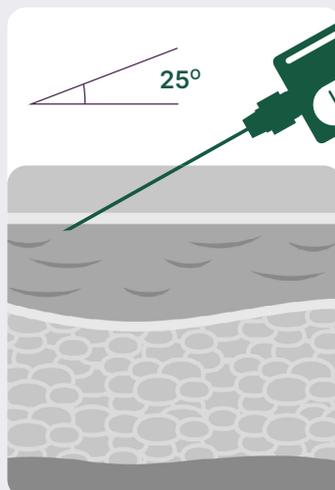

Recommended number of treatments: 2 - 3



Interval between treatments: 1 month



Shelf Life: 36 months



Storage: Store at room temperature (1-30°C), avoid direct sunlight

HILO 32 PLUS

NON CROSS-LINKED H-HA & L-HA


Package: 2.5 ml per syringe

Does not include needle

Hilo 32 Plus is designed for comprehensive mesotherapy, providing excellent benefits across multiple areas.

While its hydration and lifting effects are strong, its true strengths lie in enhancing skin elasticity, reducing inflammation, and promoting skin regeneration.

This makes it ideal for users looking for a well-rounded product that addresses various skin concerns, including aging and sensitivity.

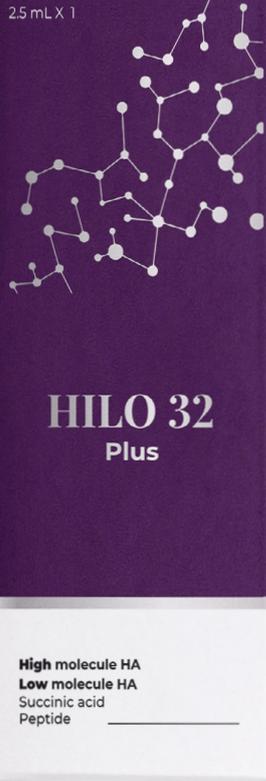

Hydration



Lifting



Volume



Elasticity



Balance Recovery



Skin Regeneration



Recommended number of treatments: 2 - 3



Interval between treatments: 1 month



Shelf Life: 36 months



Storage: Store at room temperature (1-30°C), avoid direct sunlight

Composition:

40 mg of High Molecular HA

40 mg Succinic acid



40 mg of Low Molecular HA


Peptide



Injection Depth:

Epidermis, Dermis

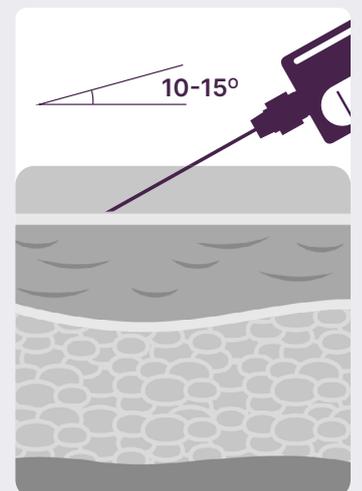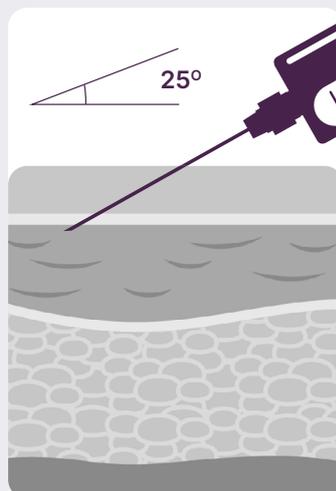

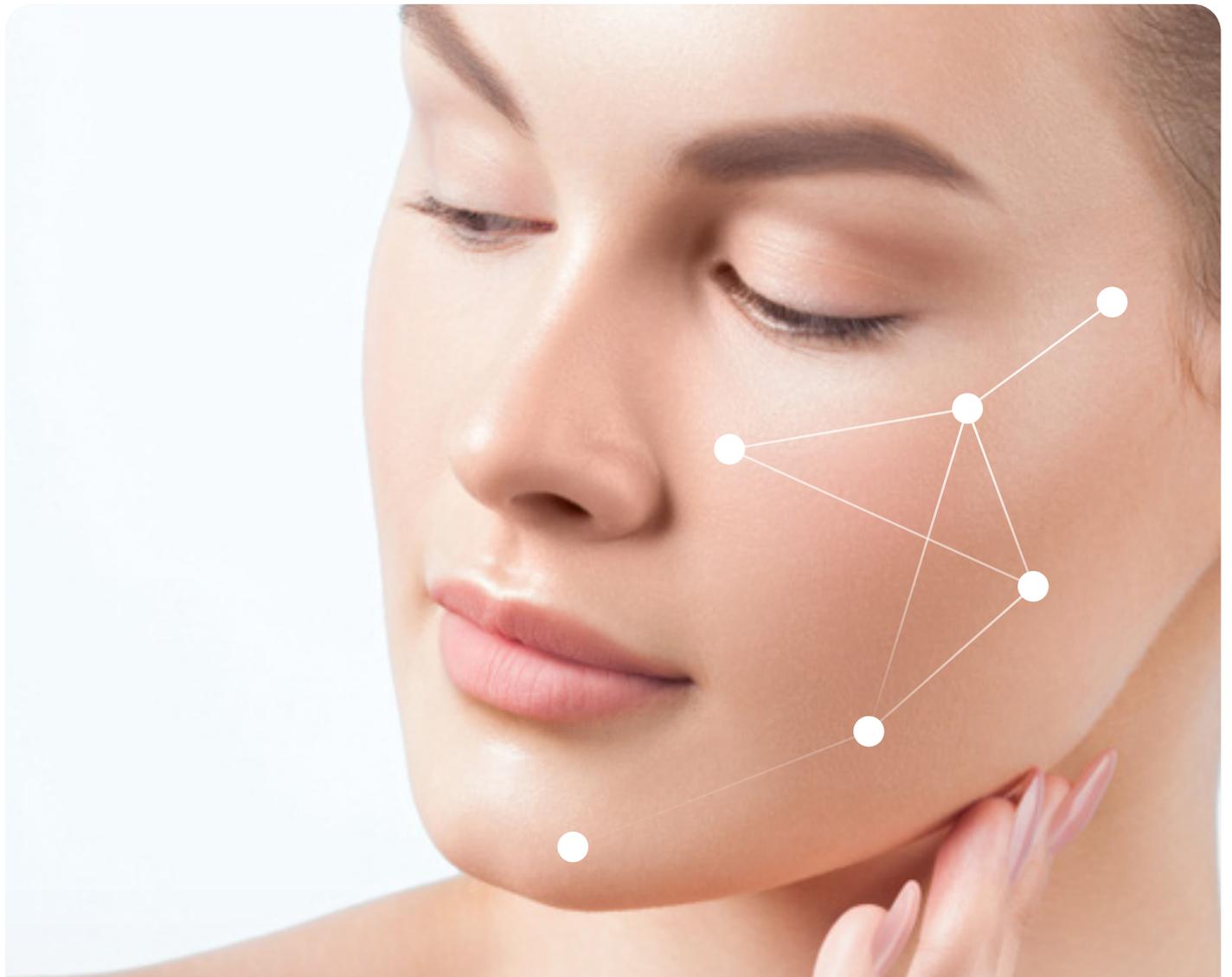

FACIAL REMODELING 6 POINTS BAP TECHNIQUE

With just **6 BAP injection points per side** of the face, **Hilo32/Hilo32 Plus** diffuses through multiple skin layers, covering broad areas of the face, and delivering a natural and long-lasting refreshed look.

Recommended Needle size: 27G / 30G

Product quantity per Point: 0.2 ml

Product quantity per Treatment: 2.4 ml


Injection depth: 1.5 - 4 mm

Angle of Inclination: 0 - 25°

Interval between treatment: 4 weeks.

Frequency: 2 -3 times per year

Avoid blood vessel locations

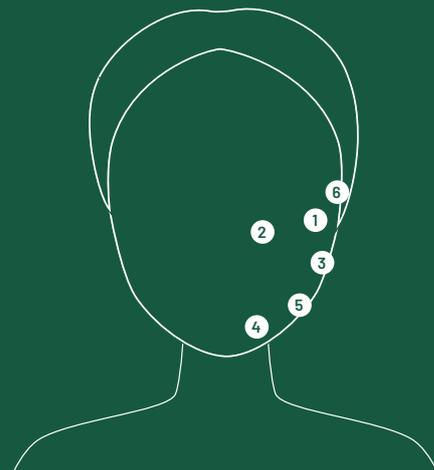

FACIAL BAP TECHNIQUE MEASUREMENTS



1. ZYGOMATIC PROTRUSION

- Identify the zygomatic protrusion at least 2 cm away from the external corner of the eye




2. NASAL BASE

- Draw a line connecting the nostril and tragus
- Draw a perpendicular line starting from the pupil
- Locate the injection point at the intersection of the 2 lines



3. TRAGUS

- Locate the tragus and mark a point 1 cm anterior to the bottom of the tragus.
- This point serves as a reference for specific treatment related to the tragus area

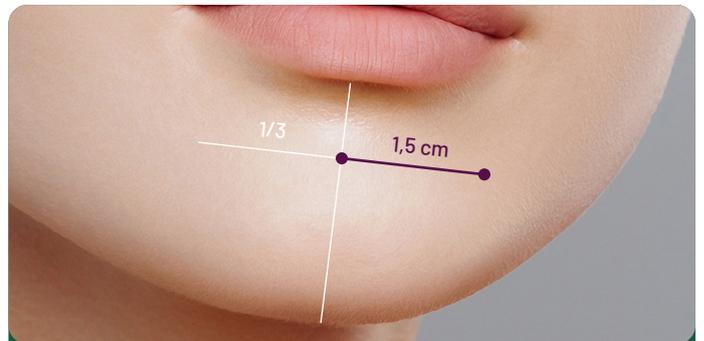

4. CHIN

- Draw a vertical line in the center of the chin
- Draw a perpendicular line one third from the top of the vertical line
- From the point of intersection move 1.5 cm towards the oral commissures.
- This precise marking assists in determining specific treatment areas on the chin



5. MANDIBULAR ANGLE

- Locate the injection point 1 cm above the mandibular



6. TEMPORALIS


- Locate the injection point 4.5 cm from the zygomatic arch



NECK REMODELING 12 POINTS BAP TECHNIQUE

The **12 BAP injection points for the neck** are carefully designed to avoid injury to vital structures, delivering anti-aging benefits from the inside out.

- Recommended Needle size:** 27G / 30G
- Product quantity per Point:** 0.2 ml
- Product quantity per Treatment:** 2.4 ml
- Distance 1.5 cm from the center of the clavicle**
- Injection depth:** 1.5 - 4 mm
- Angle of Inclination:** 0 - 25°
- Interval between treatment:** 4 weeks
- Frequency:** 2 -3 times per year
- Avoid blood vessel locations**

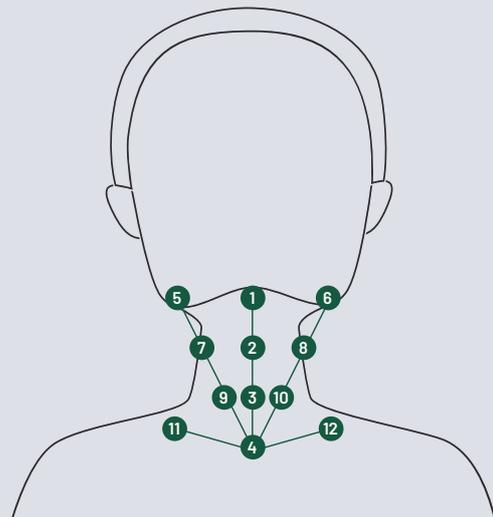



HAND REMODELING FANNING 10 POINTS BAP TECHNIQUE

The **10 BAP injection points for the hands** are meticulously placed to ensure safety and effectiveness, delivering comprehensive anti-aging benefits and rejuvenation from within.

Recommended Needle size: 27G / 30G

Product quantity per Point: 0.2 ml

Product quantity per Treatment: 4 ml (both hands)

Distance 1.5 cm from the middle of the middle finger


Injection depth: 1.5 - 4 mm


Angle of Inclination: 0 - 25°

Interval between treatment: 4 weeks

Frequency: 2 -3 times per year

Avoid blood vessel locations

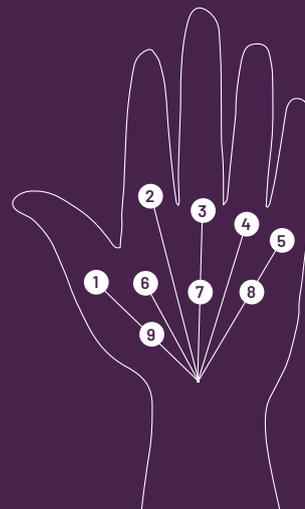

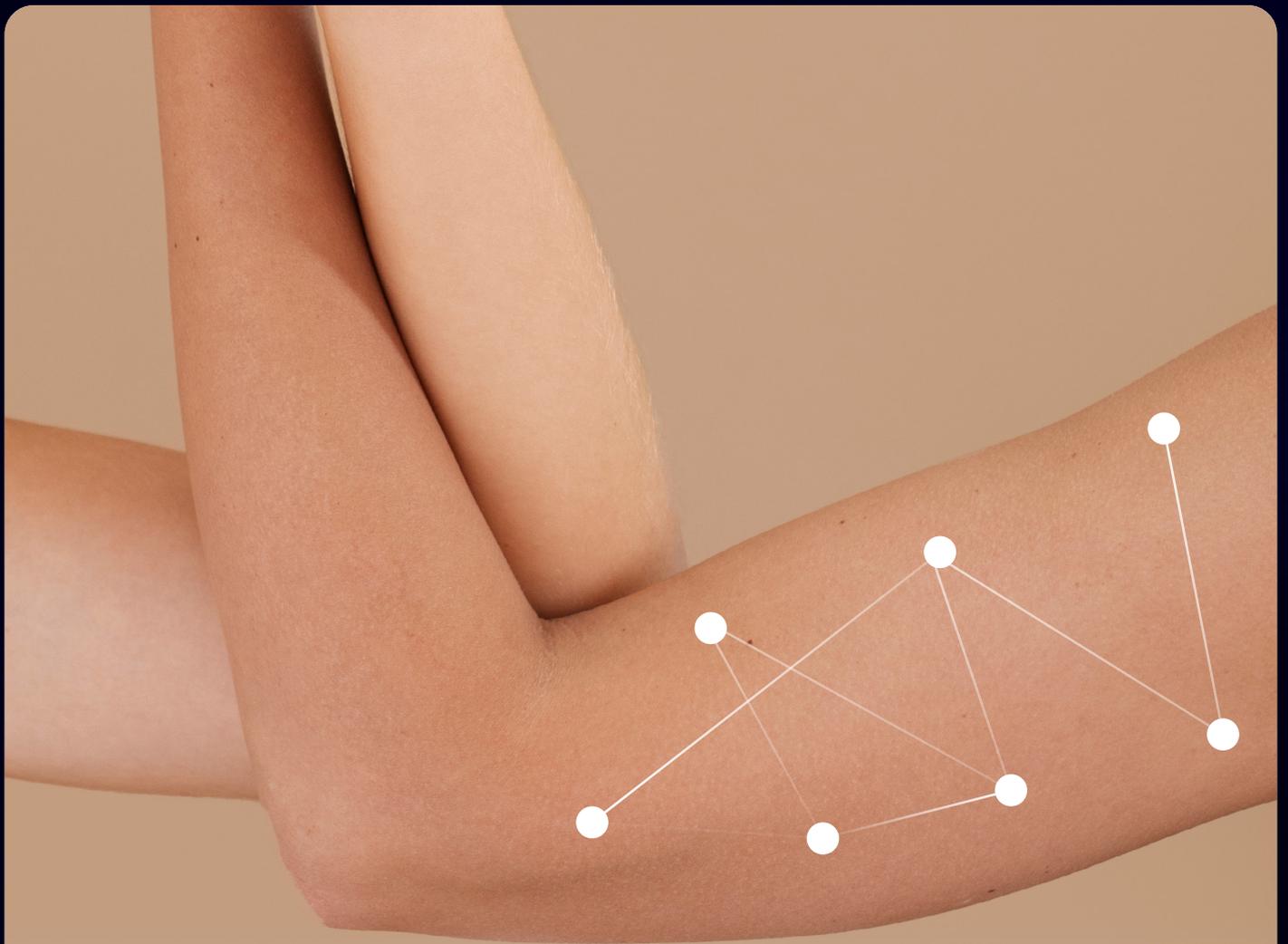

INNER ARM

7 POINTS BAP TECHNIQUE

With **7 carefully selected BAP injection points**, the inner arm receives a concentrated treatment that promotes skin rejuvenation and smoothness, effectively addressing signs of aging in this delicate area.

Recommended Needle size: 27G / 30G

Product quantity per Point: 0.2 ml

Product quantity per Treatment: 3 ml (both arms)


Injection depth: 1.5 - 4 mm

Angle of Inclination: 0 - 25°

Interval between treatment: 4 weeks

Frequency: 2 -3 times per year

Avoid blood vessel locations

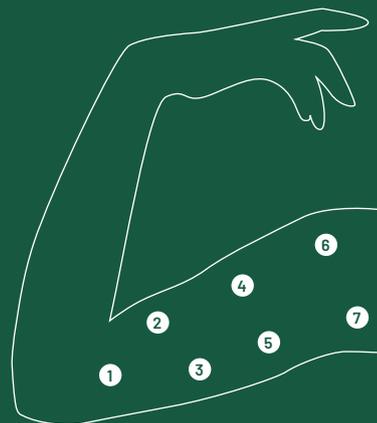

HIGH AND LOW MOLECULAR WEIGHT HA

THE PERFECT DUO
FOR SKIN PERFECTION





NOVAKOR
GLOBAL



NOVAKORGLOBAL.COM



INFO@NOVAKORGLOBAL.COM

MADE IN KOREA